Ref. 080097 / Ref. 080098

ES	3	Manual original
EN	9	
FR	15	

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CAUTION!
Diamond discs are very heat-sensitive: Do not trim or rectify metal pieces or coatings.

Ref. 080097 / 080098 is an easy to handle model trimmer specially designed for the field of dental prosthesis.

Its 3000 r.p.m. powerful motor has a brake that stops the spin after turning the main switch off.

Its dimensions give the unit an excellent stability during operation. It features a 114 mm window.

Ref. 080097 has a factory installed sandpaper disc, (80-grit). You will also receive another replacement.

Ref. 080098 has a factory installed diamond disc.

Replacements:

80-grit disc (coarse)	Ref. 080097-08 (5 u.)
120-grit disc (fine)	Ref. 080097-09 (5 u.)
Diamond disc	Ref. 080098-01 (1 u.)

The following items are also factory supplied:

- A water inlet hose.
- An extensible drainpipe.
- A 90° elbow.
- An Allen spanner to dismount the plate.

NOTE: Carefully read this manual. It contains important safety information in the installation, use and maintenance. If the equipment is used in a manner not specified by the manufacturer, the security provided by the equipment may be compromised.

MEANING OF THE SYMBOLS ON THE UNIT

Danger. Risk of injury.

Read completely these instructions before operating the unit.

Electric shock risk.

Wear safety goggles.

Wear safety gloves

Wear a mask.

INSTALLATION

Environmental conditions:

- Use indoors only.
- Environmental temperature: 5-40 °C [41-104 °F]
- Max. relative humidity: 0-95 %
- Degrees of protection: IP20

1. Carefully unpack the model trimmer. Make sure the unit has not suffered any damage during the transport.
2. Place the unit on a flat and firm surface, away from any heat or vibration source. The place has to be close to a water source with a threaded connection and to a drainpipe.
3. Connect the bent end of the flexible hose to the fitting placed at the rear of the unit. Connect the other end to the water inlet. To do this, insert the threaded tap connector at the end of the hose into the tap.
4. Connect the water outlet hose into the water outlet (D) in the right bottom front part of the unit. Connect the other end of the hose to a drainpipe. Place the hose with a certain slope, to prevent the water flow to stop and to ease the drainage.
5. For an optimum operation of the unit Talleres Mestraitúa, S.L. strongly recommends to use the Ref. 080085 plaster decanter.
6. The unit has four rubber pads. When designing the unit, special care has been taken to give the unit enough weight and robustness to prevent any movement during its operation. No additional fixing element is needed.
7. Connect the unit to a proper grounded power source.
8. Make sure that the disc cleaning system works properly. To do this, turn on the main switch in the upper part of the unit. The trimmer will start up and the electro-valve that monitors the inlet water flow will open.
9. Then open the water stopcock and select the desired flow by means of the tap knob of the trimmer. After a few minutes of operation, make sure there is no leak in the pipe and hose connections.

OPERATION OF THE UNIT

1. Depending on the characteristics of the work, adjust the tilting support surface (B) by choosing the appropriate position of the support.

Do not reposition the work table while the motor is running. This could cause it to jam on the disc and cause breakage.

2. Protect yourself properly:
 - Wear safety goggles.
 - Wear safety gloves.
 - Wear a mask.
 - Wear appropriate clothing. Do not wear loose clothing, ties, necklaces, rings, bracelets, or anything that could get caught in the trimmer.
 - Tie back long hair.
3. Turn on the main switch located on the top of the appliance (F). The motor will start.
4. Open the main water supply tap. Then, operate the trimmer's tap control (E), adjusting the desired water flow.
5. Place the model to be trimmed on the tilting support surface (B). For greater trimming precision, the support surface has marks that serve as a reference for estimating the model's angle of attack.
6. When finished, turn off the main switch.
7. After using the trimmer, it is recommended to clean the support surface and the front casing with a cloth dampened with water to remove any plaster residue.

DISC REPLACEMENT

When the abrasive element tears or wears out it must be replaced. **MESTRA** spare-parts should be always used, since they are the only ones that assure a perfect balance of the unit and top quality. To replace the disc, proceed as follows:

1. Open the lock (F) and the front cover. The safety micro-switch will prevent an eventual start-up of the motor.
2. Remove the worn out disc and place a new one. Make sure it is properly centered.
3. Close the cover again, making sure the lock (F) is properly closed.

MAINTENANCE AND CLEANING

To ensure a correct, reliable and durable performance of the unit we strongly recommend:

- Diamond discs are very heat-sensitive: Do not trim or rectify metal pieces or coatings.
- After using the unit, clean the dust and plaster remains from the table (B) and the front cover.
- Do not use alcohol or any other solvent to clean the unit.
- Never immerse the unit in a liquid.
- If you notice unusual noises or vibrations, send the unit to a technical assistance service.
- Replace the abrasive disc whenever you notice a significant fall in performance.
- After replacing the disc, check that there are not unusual vibrations.
- The unit has been designed to operate correctly leaning on the four rubber blocks in its base. These elements are strong enough to bear the movement of the trimming.

PRECAUTIONS

- ◆ Before connecting the unit to the power source, make sure the voltage is correct.
- ◆ When trimming a model, protect yourself suitably. Always wear gloves, safety goggles, and a mask. Use proper clothes. No baggy clothes, ties, necklaces, rings, bracelets or elements that could be hooked by the machine. Be careful with long hair.
- ◆ Always use the wateRef. cleaning system to clean the discs.
- ◆ Do not allow children or unskilled personnel to operate the unit.
- ◆ Do not allow anybody to by-pass the safety micro-switch that detects that the cover/door is closed or to operate the unit when the cover/door is open.
- ◆ If the motor shaft jams during normal operation, disconnect the unit from the power source before trying to free the shaft by forcing its rotation manually.
- ◆ Do not polish, brush, or grind materials that could release particles that are toxic when inhaled.
- ◆ Before using the unit, make sure it is placed on a firm and solid base.
- ◆ Do not use the unit close to solvents or other inflammable materials.
- ◆ When replacing the disc, make sure that the sprinkler (I) is in good condition and free from plaster.
- ◆ When installing the unit, make sure that the disc-cleaning circuit is perfectly leak-proof.
- ◆ Do not trim materials for which the unit has not been designed, especially metal objects.
- ◆ Do not handle or adjust the table when the unit is working.

SPECIFICATIONS

Power:	1600 W / 2 HP
Speed:	3000 rpm
Voltage:	230 V, 50/60 Hz
Fuse:	Cristal 15 A, 110-240 V
Sound power:	< 70 dBA
Abrasive disc diameter:	235 mm
Height:	340 mm
Width:	285 mm
Length:	380 mm
Weight:	10 kg

WARRANTY CONDITIONS

Requirements for the validity of the warranty

- All **MESTRA** machines are guaranteed against any manufacturing defect for one year from delivery.
- Delivery is considered to have been made from the moment the product is made available to the customer by the carrier.
- To service any warranty, it will be an essential requirement to provide the purchase invoice.

Warranty exclusions

The device has been designed and built for use by professionals in a dental clinic, dental laboratory or jewelry workshop. No other application is foreseen by the manufacturer, who consequently rejects all responsibility for damage of any kind caused by improper use of the device.

Maintenance work specific to each device, regardless of its frequency, is not covered by the warranty.

The following situations are excluded from the guarantee:

- Parts subject to natural wear (spray nozzles, tubes, lamps, filters, thermocouples, batteries, etc.).
- Breakdowns caused by physical breakage such as casings, plastics, paints, enamels, glass and similar.
- Breakdowns caused by fire, flood, weather agents, impact, crushing, inadequate ventilation or application of inadequate voltage or energy.
- Breakdowns caused by misuse by not following the safety, installation, operation, handling, cleaning, maintenance or installation instructions that are attached with each device.
- Breakdowns caused by the use of inappropriate or non-original accessories or spare parts.
- Appliances that do not have an identified factory number or that have been altered or deleted, or that do not match the number expressed on the invoice.
- Devices that have been modified, repaired or manipulated by personnel not authorized by **MESTRA**.
- Natural wear and tear from use does not represent any defect.
- **MESTRA** is not responsible for any damage that a device may cause to external elements.
- **MESTRA** reserves the right to modify the design, construction and appearance of its products without prior notice.
- All demands that exceed the limits of this guarantee are excluded, especially demands for compensation due to loss of profit, compensation for use, as well as indirect damages, as long as they are not mandatory liability according to law.

Damage caused by transportation

- In cases where transportation is the responsibility of **MESTRA**, the customer must examine the product upon receipt. If any damage has occurred due to transport, the client must immediately note it on the carrier's delivery note and notify **MESTRA**. Otherwise, **MESTRA** will not be able to claim from the carrier.

Repair conditions

MESTRA will repair or replace the machine as it deems appropriate, under the conditions stipulated below.

- Due to this guarantee, our company is only responsible for repairing or replacing the product or part recognized as defective and returned following the agreed procedure.
- During the period of repairs or changes no equivalent equipment will be supplied.
- Defective parts or components removed will become the property of **MESTRA**.

Repairs outside the mestra facilities

- **MESTRA** will not carry out any repairs outside its workshop and is exempt from the transportation costs this may cause.
- Repairs made at the customer's home, whether under warranty or not, are entrusted to the technical service of the distributor that made the sale or to its contracted technicians.

Send conditions

- Any product return invoking the guarantee must be previously notified and accepted. To this end, the customer will contact our after-sales service.
- The defective product will preferably be sent in its original packaging or, failing that, perfectly packaged.
- The costs and risks derived from transporting the product are the responsibility of the client and these will always be carried out through a courier company so that the merchandise is insured against possible damage during the journey.
- The equipment to be repaired must be accompanied by a note indicating the defect observed, name, address and telephone number of the user. To speed up the process, it is necessary to add an exact description of the defect.

If the client requests the replacement of a component under warranty, the amount will initially be charged. The customer will send the defective component to the factory for review and payment of the amount.

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