Ref. 100236

PRIME 221

ES.....	3	Manual original
EN.....	7	
FR.....	11	

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
Introduction

The PRIME 221 is a handpiece/control unit combination specifically designed for use in the dental prosthetics, jewelry, and podiatry industries. To ensure a long and satisfactory product life and, above all, safe use, we recommend that you read these instructions carefully.

MESTRA is not responsible for any problems arising from failure to follow these instructions. We also remind you that ALL MESTRA products are intended for use by properly trained professionals.

Description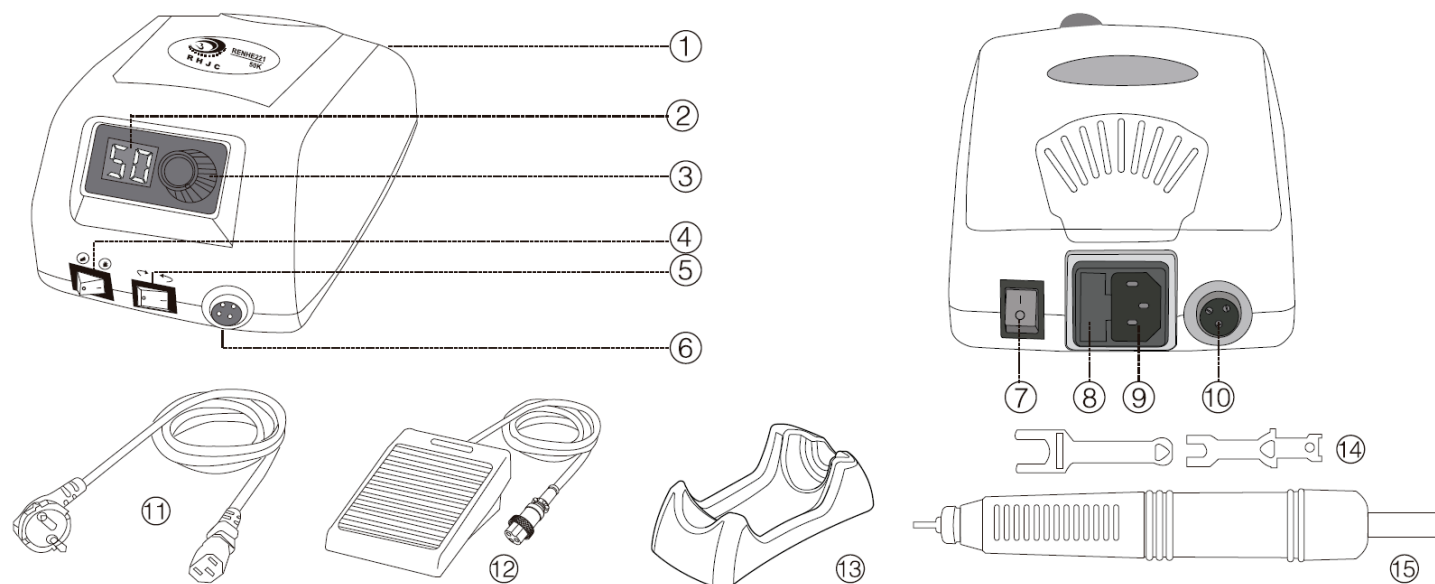

- 1) Control unit
- 2) RPM display
- 3) Speed control dial
- 4) Manual/foot pedal button
- 5) Forward/reverse button
- 6) Handpiece connector
- 7) Power switch
- 8) Fuse holder

- 9) Power connection
- 10) Foot pedal connector
- 11) Power cord
- 12) Handpiece holder
- 13) Speed control pedal
- 14) Keys
- 15) Handpiece

Contents and technical data

The set associated with PRIME 221 consists of the following elements:

▪ Electronic control unit


Power supply: AC200~240V.

Fuse: 2 A

Weight: 1,49 kg

Maximum current: 5 A

Dimensions: W 130 x L 191 x H 90 mm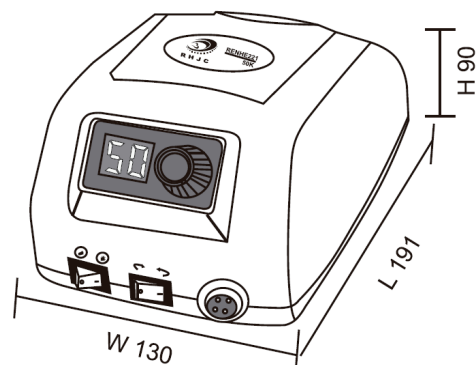

▪ Handpiece


Power: 120 W

Maximum speed: 50 000 rpm

Torque: 5,8 N·cm

Weight: 205 g

Dimensions: Ø 20-26 x L 157 mm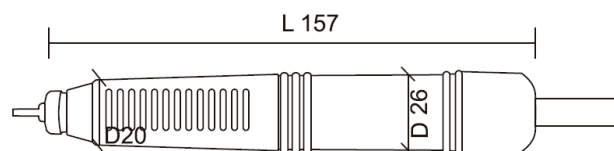


▪ Accessories

Included in this section is the power cord, foot pedal, a rubber handpiece holder, and two handpiece disassembly tools.

IMPORTANT

For proper use of the micromotor and to ensure its proper function and durability, it is important that the user properly adjust the RPM to the type and diameter of the bur used at all times. A bur rotating at the incorrect speed will produce poor-quality work, in addition to affecting the device's lifespan and user safety. Depending on the type of bur used, the user must select the maximum operating speed range to avoid accidentally exceeding this limit.

The maximum recommended speed for each bur depends on many parameters; therefore, the user must select it based on their experience and professional knowledge. As a guide, we present the following table with maximum RPM recommendations based on the bur type and diameter. Under no circumstances should this guideline be taken as a guideline that guarantees proper use and operation. If in doubt, consult the manufacturer of the bur you wish to use.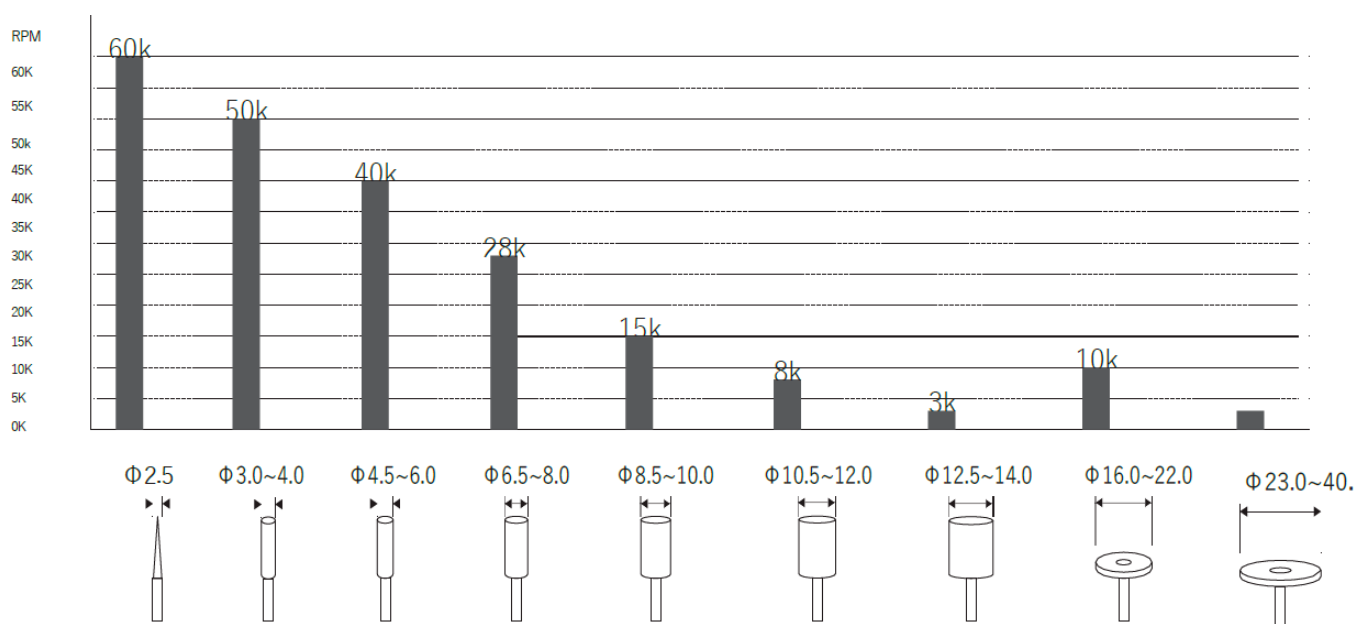

(mm)

Installation

1. Carefully unpack the device. Check that the set includes all the items described in the "Description" section.
2. Choose a flat, stable location, away from all sources of heat, vibrations, or direct sunlight. The location should also be away from damp areas and flammable substances.
3. Also ensure that the unit's ventilation grilles (located on the top of the control unit) are perfectly clean and at least 20 cm away from any obstacles.
4. Connect the foot pedal to the control unit connector. Also connect the handpiece to the connector.
5. Make sure the power switch is in the "OFF" position. Connect the power cable to the mains connector and then to the mains power supply. It is important to ensure that the power outlet is 230 V, 50 Hz.
6. After these steps, the device is ready for use.


IMPORTANT: The operations of connecting the pedal, the handpiece or the power cable must always be carried out with the unit disconnected from the mains (switch in OFF)

Use

1. With the device installed according to the instructions in the previous point, press the switch to the ON position (1).
2. The display will turn on.
3. Select the parameters according to your needs:
 - Direction of rotation
 - Speed (rpm)
 - Operating mode (manual/pedal).
4. If in "manual" mode, pressing the start/stop button will start the handpiece rotating. The speed can be changed by pressing the round knob. Pressing the start/stop button again will stop the handpiece.
5. If in "pedal" mode, pressing the pedal will start the handpiece rotating. The rotation speed can be adjusted by adjusting the depth of the pedal.
6. **It is important to choose the right type of bur for each job, as well as the appropriate rotation speed.**

Mounting the bur on the handpiece

1. Firmly grasp the lower half of the handpiece (the part where the connection cable exits).
2. At the same time, hold the upper half of the handpiece (the part where the bur is mounted) with your other hand.
3. To unlock an already mounted bur, turn the top of the handpiece counterclockwise. When it reaches the stop, you will feel a distinctive "click." The bur shaft is released.
4. To mount a new bur, simply insert the bur shaft into the collet. The recommended insertion distance for the bur shaft is approximately 27/30 mm.
5. Then, turn the upper part of the handpiece clockwise until it stops. When it reaches the stop, you will also feel a distinctive "click."
6. Make sure the collet grips the bur shaft properly and that the micromotor and bur shafts are perfectly aligned. An improperly secured or misaligned cutter can cause a serious accident.
7. Perform cutter changes with the unit disconnected from the power supply.

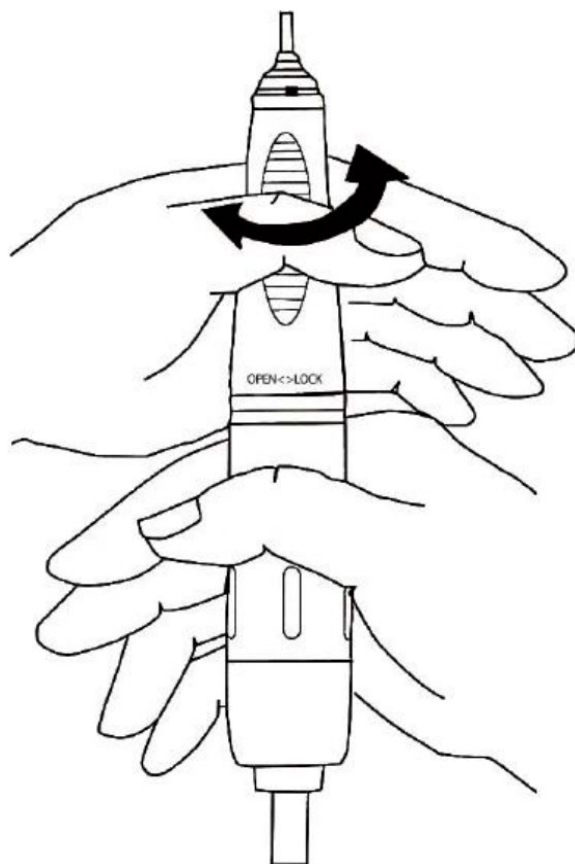Precautions

- Protective goggles, a mask, and gloves must be worn when operating the device.
- Do not use the device if you notice any fault or defect. If in doubt, consult MESTRA Technical Support.
- Always check the device's cooling system: the control unit must have its ventilation vents clear and be more than 20 cm away from any obstacle.
- Periodic cleaning and maintenance are required for this device. If in doubt, consult your MESTRA Technical Support.
- The device has been designed to be operated by a specialist technician duly qualified in dental work and risk prevention.
- Ensure the power switch is off before connecting to the power supply, foot pedal, or handpiece. This device is also required during bur changes and during maintenance and cleaning.
- Do not force the device. Try to work with the lowest possible RPM range and torque. If in doubt, consult the bur manufacturer.
- Do not open the bur holder while the handpiece is in operation. The control unit must be turned off to perform this operation.
- Do not leave the handpiece on the table or on its stand until it has come to a complete stop.
- Make sure to unplug the power cord before moving the control box.
- Use the specified voltage and frequency: 230 V, 50 Hz.
- Use only accessories suitable for this device. If in doubt, consult your MESTRA technical service.
- Always use a suction unit suitable for the device's specifications. Operation must be synchronized.
- The selected parameters (direction of rotation, manual/pedal, RPM) are saved in memory when the device is turned off. Be careful when turning it back on.
- Do not work in hazardous environments (humidity, flammable materials, etc.). Do not place the control unit near a damp location.
- Check proper operation of the equipment, including anchoring and vibration of the cutter, at low speed and before starting high speed.

Cleaning and maintenance

- Keep the area around the micromotor free of dust and work debris.
- Remove dust from the handpiece by wiping it with isopropyl alcohol and avoiding getting the interior wet. Do not apply oil or clean the unit with water. Grease inside the bearings could leak out due to the oil.
- Air cleaning is acceptable, but compressed air could adversely affect the handpiece.

Problems and solutions

Problem description	Check	Solution
Display does not turn	Power supply, fuse, switch	Try changing the plug or cable.
Error codes	E1/E5 Hardware protection	Replace the electronic control.
	E2/E4 Overload protection	Turn the power off and on again.
	E3 Voltage protection	Check that the voltage is correct.
	E6 Temperature protection	Check if the handpiece has been in use for a long time.
Buttons do not work	Check if a beep is heard	Consult your dealer.
Handpiece gets hot	Handpiece is warm	It has become hot due to continuous use. Let the handpiece rest until it cools.
	Handpiece is hot	It has overheated for an excessive amount of time.
Handpiece does not rotate	Check if the collet is open (E4)	Close the collet.
	Check if the handpiece is properly connected (E4)	Connect the handpiece correctly. If the error persists, contact your dealer.

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